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(0,15 % Hyaluronate de sodium)

Collyre

DESCRIPTION

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Eye drops

DESCRIPTION

BIOLAN® is a sterile, preservative-free, elastoviscous, clear corneal shield. It consists only of natural biological substances. **BIOLAN®** protects, moistens and lubricates the surface of the eye for prolonged relief of discomfort associated with • mechanical stress, e. g. during wearing of hard or soft contact lenses, or during ophthalmic diagnostic procedures • environmental stress, e. g. induced by air conditioning, wind, coldness, dryness or air polluted by cigarette smoke for example • visual stress from intensive viewing, e. g. at computer terminals, microscopes or during prolonged operation of motor vehicles.

APPLICATION

Detach one unit dose container. Make sure that the unit dose container is not damaged. Twist off protective cap (do not pull). Apply one or two drops to each eye or to inner surface of the contact lens. If you experience permanent discomfort, please consult your ophthalmologist!

COMPOSITION

0.15 % sodium hyaluronate, sodium chloride, sodium phosphate monobasic, sodium phosphate dibasic, purified water.

PACKAGE SIZE

Commercial unit: Unit dose containers,
20 x 0.35 ml
60 x 0.35 ml

PRECAUTIONS AND WARNINGS

- Do not use the product if unit dose container is damaged • Do not touch the eye with the unit dose container • Do not allow any contact between the opening of the unit dose container and other surfaces to avoid risk of contamination.

- For single use only; as **BIOLAN®** does not contain preservatives, any solution not used immediately after opening should be discarded • Do not use the product together with topical ophthalmic drugs • Do not use in infected eyes • Temporary blurring for a brief period may be experienced after application; avoid driving, operating machinery and wait until normal vision is restored • Do not use the eye drops beyond the expiry date which you can find on each unit dose container.

STORAGE

Store at 2 °C to 25 °C. Do not freeze. 

Lotnumber  and date of expiry  imprinted on each unit dose container

STERILE  

 **Manufacturer:**
Penta Arzneimittel GmbH
Werksstraße 3
92551 Stulln
Germany

Distribution in Germany:
Santen Oy
Niittyhaankatu 20
33720 Tampere,
Finland
Contact:
Santen GmbH
Industriestrasse 1
82110 Germering
Germany

 0483