Information for wearers of contact lenses

CONTOPHARMA GPHCL Moisturizing M (part of the contact lens care system CONTOPHARMA GPHCL)

This solution helps rewetting rigid, gaspermeable contact lenses.

Warning:

In order to avoid complications in the system eye – contact lens, the contact lens care products should only be used after having consulted a contact lens practitioner and under consideration of the used contact lens material. The use of not properly cleaned or damaged contact lenses can lead to serious damage to the eye or eyesight. The solution may not be used, if there is an allergy to any of its contents. In case of redness of the eye in the area of touch of the contact lens or the conjunctiva, you should immediately contact your ophthalmologist or contact lens practitioner.

Special indications for use:

- After the bottle has been opened, the solution may be used for no longer than 60 days (note the date of first use on the bottle). The remaining contents after 60 days must be disposed of professionally.
- Store the solution at room temperature (15° – 25°C) and keep out of direct light.
- Observe general rules of hygiene such as washing the hands before handling the lenses.
- In order to avoid a possible contamination by microorganisms of the care product, you should not touch the dropper of the bottle. Close the bottle promptly after use and always keep it well closed.
- A regular and correct care of your contact lenses increases the wearing comfort and reduces complaints on the eye that might arise as a consequence of an improper care.

Directions for use:

In order to avoid eventual restrictions of compatibility during wear you can rewet the lenses with CONTOPHARMA GPHCL Moisturizing M. Thus you will get an optimal wearing comfort.

- Put 2 to 3 drops of CONTOPHARMA GPHCL Moisturizing M into the external palpebral angle of the eye once or several times a day, as necessary.
- Do not touch the eye with the dropper of the bottle.

So far no side effects could be observed when using CONTOPHARMA GPHCL Moisturizing M according to instructions.

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Distributeur: Contopharma AG
Distributore: Contopharma AG