Anwenderhinweise zur Contopharma Comfort-Lösung «drop & see»

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Contopharma AG
CH-3800 Interlaken
www.contopharma.ch

Information for users of the Contopharma «drop & see» Comfort Solution

Specially designed wetting solution with optimal protection for the eyes.

In clinical trials of practical suitability it was found to be most suitable for all common gas-permeable, flexible hard lenses and all common hydrophilic soft lenses, especially when changed regularly (1–3 months).

For whom is «drop & see» suitable?

The solution is recommended as a preventive measure for all persons and especially those with dry or slightly irritated eyes that may be stressed by environmental inconveniences such as climatisation, wind, dryness, air pollution, prolonged working at a computer screen or wearing contact lenses for long periods.

Instructions for use

As desired, drop the solution into the outer angle of the eyelids. It may be used several times daily, even while still wearing contact lenses.

Warnings

- The solution must not be used if you are allergic to any of the ingredients. If the conjunctiva becomes red or other eye symptoms arise, you should immediately consult an optician or a contact lens specialist.
- Do not use damaged bottles.
- After opening the container for the first time, do not use the solution for longer than 30 days (note the date of first opening on the bottle). Any solution remaining after 30 days can be disposed of as normal. The polyethylene bottle is designed to comply with the requirements of various recycling concepts.
- Store the solution at room temperature (15–25 °C) and protected from the light. To prevent possible microbial contamination of the solution, do not touch the dropper unit on the bottle. Close the bottle immediately after use and always keep it tightly closed.
- If you cannot see as clearly as normal for a short while after using the solution, you should not drive or use machines until your sight has returned to normal.
- Keep out of the reach of children.

When used as instructed, no side effects have so far been observed with CONTOPHARMA «drop & see» Comfort Solution.

Special instructions for use

The «drop & see» solution may also be used temporarily and in exceptional situations as a storage solution for all current types of contact lenses.

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Laboratorium Dr. G. Bichsel AG
Contopharma AG, CH-3800 Interlaken