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**NOVOXY SYSTEM (1&2)
DISINFECTING, CLEANING,
NEUTRALIZING, RINSING
AND STORING SYSTEM
FOR ALL TYPES OF SOFT
CONTACT LENSES.**

INSTRUCTIONS FOR USE:

1. Before handling your lenses carefully wash your hands.
2. Fill the lens case up to the mark with NOVOXY1 solution.
3. Place lenses in the lens case. Close it and shake it for a few seconds to disinfect the inside of the lens case.
4. Keep the lenses in the solution for at least 20 minutes or overnight.
5. Open the lens case and remove the solution.
6. Fill the lens case up to the mark with NOVOXY2 solution and close it. Shake for a few seconds to neutralize all the rests of NOVOXY 1 solution.
7. Keep the lenses in the NOVOXY 2 solution for at least 15 minutes and the lenses will be ready for use.

PRECAUTIONS:

- MAKE SURE NOVOXY 1 SOLUTION DOES NOT COME INTO CONTACT WITH YOURS EYES.
- Before using the lenses always neutralize them with NOVOXY 2.
- Always maintain the bottle well closed. Store at room temperature and keep out of children's reach.
- Once opened, the solution should not be used after 60 days.
- Do not touch the dropper or dirty it.
- In case of irritation, consult your contact lens specialist.

COMPOSITION:

NOVOXY 1

3% hydrogen peroxide solution

NOVOXY 2

Thimerosal 0.001 % and enzymatic catalyst in buffered, isotonic and sterile solution.

AVIZOR

La Canada, 17

28850 Torrejón de Ardoz (SPAIN)

www.avizor.com

STERILE | A

www.MrLens.ch

MrLens GmbH
Chamstr. 172
6300 Zug
Tel. 041 - 741 88 87
Fax. 041 - 741 08 89
e-Mail: info@mrLens.ch