Regard[®]

Gas permeable
contact lens solution

K

FOR THE USE AND SAFETY OF THIS PRODUCT,
PLEASE READ CAREFULLY AND KEEP THESE
PRODUCT INSTRUCTIONS.

Congratulations on choosing **REGARD K**, a scientific advancement in rigid and gas permeable contact lens care that provides superior lens comfort and hygiene. **REGARD K** contains an active disinfectant (complexed stabilized OxyChlorite[®]™ molecule) that breaks down into the components of natural tears (H₂O and NaCl) and releases oxygen. **It's kind to your eyes due to the absence of preservatives like PHMB.**

REGARD K cleans, rinses, disinfects, stores and lubricates rigid and gas permeable contact lenses and is designed to help prevent the formation of protein deposits on the lens surface without enzyme protein removers.

REGARD K is a sterile, isotonic, buffered solution that contains, purified water, sodium chloride, boric acid, hydroxypropylmethylcellulose, poloxamer and OxyChlorite[™].

Instructions for use

- Ensure that the tamper-evident seal is integral.
- **Always** wash and rinse your hands thoroughly before removing your lenses.
- Place a few drops of **REGARD K** on each contact lens and gently rub to clean the surface of your lenses.
- Rinse each lens with **REGARD K** to remove debris. Never rinse your lenses with tap water.
- Only use the original **REGARD K** lens case. Place your lenses in their lens case and fill completely to the rim with fresh **REGARD K** solution. Close tightly and turn the lens case upside down and back.
- Soak your lenses for at least six hours to disinfect your lenses and remove protein deposits. **REGARD K** destroys harmful micro-organisms and their toxins and enzymes. If you store your lenses more than thirty days, repeat the disinfection with fresh **REGARD K** solution.
- Put one or two drops of **REGARD K** onto the internal lens surface before insertion to increase the comfort upon application.
- Rinse your lens case with fresh **REGARD K** solution and allow to air dry after each use.

Precautions

REGARD K solution should never be reused. Avoid touching the bottle tip and close tightly after each use. Store at room temperature and away from direct sunlight. Do not use after expiration date and discard an opened bottle after 90 days.

As with any personal care product, discontinue if irritation occurs or you are allergic to any of the ingredients. Keep away from children. Not for internal use.

REGARD[®]K[™] and OxyChlorite[™] and the **REGARD K** logo and package design are trademarks of Vita Research. USA and European patents applied. ©Vita Research

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