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Bausch & Lomb Incorporated  
Rochester NY 14609, USA

Bausch & Lomb Incorporated  
106 London Road  
Kingston-upon-Thames KT2 6TN, UK

CE 0050

®/™ indicano i marchi registrati di Bausch & Lomb Incorporated  
© Bausch & Lomb Incorporated

**SOFLENS® DAILY DISPOSABLE****⚠ INSTRUCTIONS FOR USE**

This leaflet is intended to inform users of SofLens® daily disposable hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

<b>Product Name</b>	<b>Material</b>	<b>Water content</b>	<b>Packaging Solution</b>
SofLens® daily disposable (hilafilcon B) Visibility Tinted Contact Lenses	Hilafilcon B	59% by weight	Borate buffered saline
SofLens® daily disposable Toric (hilafilcon B) Visibility Tinted Contact Lenses	Hilafilcon B	59% by weight	Borate buffered salin

**Description and wear application**


These lenses are intended for daily disposal wear and should be discarded after each wearing period. Reuse without cleaning and disinfecting the lenses could lead to infection and/or serious eye injury. Always follow your eye care practitioner's recommended wearing schedule.

These lenses are supplied sterile and are intended by the manufacturer for a maximum wear period of up to one day. They have been sterilized by steam. **STERILE** 

If the packaging is damaged or opened DO NOT USE.

**Insertion and removal instructions**

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

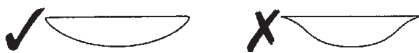Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens down to the white part of the eye and remove it. Use a rewetting drop if the lens is difficult to remove. Always discard lens following removal from the eye.

**Caring for your lenses**

- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes
- Always discard lenses after removal.

## Warnings and precautions

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Do not wear daily disposable lenses for longer than a day or while sleeping.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes.

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication.

If you experience any of the following problems, remove your lenses **immediately**:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal, cleaning and reinsertion of the lens or insertion of a new lens, then remove the lens and **contact your eye care practitioner**. These symptoms can lead to more serious conditions if ignored.

Bausch & Lomb Incorporated  
Rochester NY 14609, USA

Bausch & Lomb Incorporated  
106 London Road  
Kingston-upon-Thames KT2 6TN, UK

®/™ denote trademarks of Bausch & Lomb Incorporated.

© Bausch & Lomb Incorporated